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## EN MD 555 cleaner Special detergent for suction systems

MD 555 cleaner is a non-foaming special cleaner for dental suction systems including drainage lines. Use of MD 555 cleaner is recommended for regions with problematic tap water (e.g. high water hardness, antisoluble salts, etc.) and for drainage lines with insufficient gradient. Regular use of MD 555 cleaner prevents formation of deposits and blockages, and dissolves existing deposits. Furthermore, MD 555 cleaner is particularly indicated when special tooth cleaning equipment is used, such as air abrasion systems.

Regular application of MD 555 cleaner prevents build-up of deposits in the suction system. Blockages in the suction system due to the use of spalling soluble preventive products such as preventive powders and spray products, especially on the basis of calcium carbonates, are avoided and already existing deposits are dissolved. The special active ingredients and other constituents make MD 555 cleaner particularly suitable for systems with amalgam separators. Furthermore, MD 555 cleaner can be used anywhere where deposits of water hardness need to be removed from surfaces (such as tiles or sinks).

**Composition:** MD 555 cleaner contains organic acids (15 - 30%), phosphoric acid (15 - 30%), non-foaming surfactants (< 5%) and auxiliary substances in aqueous solution.

**Application:** In situations with hard water or prophyl powder contamination, apply a 5 percent solution of MD 555 cleaner regularly (once to twice per week) e.g. during lunch breaks. In case of extremely high levels of contamination, the solution may also be applied daily. The shortest contact time is 30 minutes. This can be extended if required. To use, make up MD 555 cleaner to a volume of 1 litre with water and suck the solution through using the OroCup. Also add 250 ml of solution to the spittoon. At the end of the standing time, flush the system after cleaning with at least 1 litre of water.

In problematic situations and when cleaning suction systems with severe biofilm contamination (in this case, alternating with active disinfectants such as Orotol® plus or Orotol® ultra, MD 555 is used - once to 5 times a week - at a concentration of 10% (make 100 mL MD 555 cleaner up to 1 litre with water). The optimum effect is obtained with a contact time of 1 - 2 hours. Also add 250 ml of solution to the spittoon.

**Please note:** Rinse suction systems thoroughly with plenty of water after use and before using the solution of MD 555 cleaner. Orotol® plus or Orotol® ultra. Make sure to avoid cleaning cycles followed immediately by disinfection cycles. MD 555 cleaner is not suitable for use on aluminium and other acid-sensitive surfaces. Do not leave MD 555 cleaner in contact with surfaces overnight.

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**Application:** In situations with hard water or prophyl powder contamination, apply a 5 percent solution of MD 555 cleaner regularly (once to twice per week) e.g. during lunch breaks. In case of extremely high levels of contamination, the solution may also be applied daily. The shortest contact time is 30 minutes. This can be extended if required. To use, make up MD 555 cleaner to a volume of 1 litre with water and suck the solution through using the OroCup. Also add 250 ml of solution to the spittoon. At the end of the standing time, flush the system after cleaning with at least 1 litre of water.

In problematic situations and when cleaning suction systems with severe biofilm contamination (in this case, alternating with active disinfectants such as Orotol® plus or Orotol® ultra, MD 555 is used - once to 5 times a week - at a concentration of 10% (make 100 mL MD 555 cleaner up to 1 litre with water). The optimum effect is obtained with a contact time of 1 - 2 hours. Also add 250 ml of solution to the spittoon.

**Please note:** Rinse suction systems thoroughly with plenty of water after use and before using the solution of MD 555 cleaner. Orotol® plus or Orotol® ultra. Make sure to avoid cleaning cycles followed immediately by disinfection cycles. MD 555 cleaner is not suitable for use on aluminium and other acid-sensitive surfaces. Do not leave MD 555 cleaner in contact with surfaces overnight.

see imprint.

Warning. Causes skin irritation. Causes serious eye irritation. Wear protective gloves and eye/face protection. If skin irritation or rash occurs: Get medical advice/attention. If IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container to hazardous or special waste collection point.

**DÜRR DENTAL (Products) UK Ltd.**
14 Linnell Way - Telford Way Industrial Estate
Kettering, Northants NN16 8PS, United Kingdom
☎ +44 1536526740

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